

## Syringe Pump Extension Set - Non PVC Fluid Path

30" Minibore extension set with slide clamp,  
male luer-lock, and female luer-lock

### FOR PARENTERAL FLUIDS

---

#### CAUTIONS:

- Do not use if package is damaged and/or protection caps are detached. Keep dry. Store at controlled room temperature. Keep away from sunlight.
- Use aseptic technique.
- Single use only.

#### MORE INFORMATION:

- Length is approximate
- Approximate contained volume 0.5 mL
- Gravity flow rate 792 mL/hr
- Sterile and nonpyrogenic fluid path
- Fluid Path Material: PE lined (tubing)(Not made with DEHP) / PP (end caps) / Copolyester (female luer) / Cyrolite (male luer)

#### CONTRAINDICATIONS:

- Do Not Use for light sensitive drugs unless indicated. Some drugs may be incompatible with tubing materials and components check drug labeling and facility protocol.

#### PRIMING:

**Priming must be completed before connecting to patient.**

- Attach syringe, prime with medication, and ensure all air is removed
- Clamp tubing
- Swab patient's access site or other luer connection before connecting to patient per facility protocol
- Remove end cap and attach to patient luer

#### SET REMOVAL:

- Stop infusion. Close slide clamp, disconnect the set from IV access and container in accordance with facility protocol. Replace per facility protocol or CDC guidelines. Discard biohazard waste according to facility protocol.

**Manufactured For:**  
Fresenius Kabi AG  
Else-Kröner-Str.1  
61352 Bad Homburg, Germany  
[www.fresenius-kabi.com](http://www.fresenius-kabi.com)

**Distributed in USA by:**  
Fresenius Kabi USA, LLC  
Three Corporate Drive  
Lake Zurich, Illinois 60047  
[www.fresenius-kabi.us](http://www.fresenius-kabi.us)  
Tel.: 1-800-933-6925

N. Code No. : HR/Drugs 737-B(H)

10.5520.00

10.5952.01