

Syringe Pump Extension Set

60" Macrobore extension set with slide clamps,
male luer-lock, and female luer-lock

FOR PARENTERAL FLUIDS

CAUTIONS:

- Do not use if package is damaged and/or protection caps are detached. Keep dry. Store at controlled room temperature. Keep away from sunlight.
- Use aseptic technique.
- Single use only.

MORE INFORMATION:

- Length is approximate
- Approximate contained volume 3.0 mL
- Gravity flow rate 24,000 mL/hr
- For gravity use, flow rate is regulated by roller clamp
- Sterile and nonpyrogenic fluid path
- Fluid Path Material: PVC (Not made with DEHP) / PE (end caps) / PC (luer locks)

CONTRAINDICATIONS:

- Do Not Use for light sensitive drugs unless indicated. Some drugs may be incompatible with tubing materials and components check drug labeling and facility protocol.

PRIMING:

Priming must be completed before connecting to patient.

- Attach syringe, prime with medication, and ensure all air is removed
- Clamp tubing
- Swab patient's access site or other luer connection before connecting to patient per facility protocol
- Remove end cap and attach to patient luer

ATTACHING TO NEEDLELESS CONNECTORS:

- Swab surface of the needleless connector prior to and after every use according to CDC guidelines or facility protocol. Use luer lock connections only.

SET REMOVAL:

- Stop infusion. Close slide clamp, disconnect the set from IV access and container in accordance with facility protocol. Replace per facility protocol or CDC guidelines. Discard biohazard waste according to facility protocol.

Manufactured For:
Fresenius Kabi AG
Else-Kröner-Str.1
61352 Bad Homburg, Germany
www.fresenius-kabi.com

Distributed in USA by:
Fresenius Kabi USA, LLC
Three Corporate Drive
Lake Zurich, Illinois 60047
www.fresenius-kabi.us
Tel.: 1-800-933-6925

N. Code No. : HR/Drugs 737-B(H)

10.5520.00

10.5954.01