

## Stopcock

4-Way stopcock with rotating luer-lock

### FOR PARENTERAL FLUIDS

---

#### CAUTIONS:

- Do not use if package is damaged and/or protection caps are detached. Keep dry. Store at controlled room temperature. Keep away from sunlight.
- Use aseptic technique.
- Single use only.

#### MORE INFORMATION:

- Length is approximate
- Approximate contained volume 0.45 mL
- Gravity flow rate 8,580 mL/hr
- Sterile and nonpyrogenic fluid path
- Fluid Path Material: Non-PVC and Non-DEHP HDPE (4-way stopcock)

#### CONTRAINDICATIONS:

- Materials of the the gravity administration set does not provide light protection.

#### PRIMING:

**Priming must be completed before connecting to patient.**

- Connect to primary set
- Tighten all connections
- Before use, ensure that the stopcock is properly primed to remove any air bubbles
- Remove white cap(s) before connecting syringe or tubing
- Replace white cap per hospital policy
- Clamp set
- Swab patient's access site or other luer connection before connecting to patient per facility protocol
- Remove end cap and attach to patient luer
- To use stopcock, align arrows on handles with desired flow path

#### SET REMOVAL:

- Stop infusion. Close slide clamp, disconnect the set from IV access and container in accordance with facility protocol. Replace per facility protocol or CDC guidelines. Discard biohazard waste according to facility protocol.

**Manufactured For:**  
Fresenius Kabi AG  
Else-Kröner-Str.1  
61352 Bad Homburg, Germany  
[www.fresenius-kabi.com](http://www.fresenius-kabi.com)

**Distributed in USA by:**  
Fresenius Kabi USA, LLC  
Three Corporate Drive  
Lake Zurich, Illinois 60047  
[www.fresenius-kabi.us](http://www.fresenius-kabi.us)  
Tel.: 1-800-933-6925

N. Code No. : HR/Drugs 737-B(H)

10.5520.00

10.5963.01