

Short Extension T-Connector

6" Minibore extension set, bonded K-Zero T-Connector,
and slide clamp

FOR PARENTERAL FLUIDS

CAUTIONS:

- Do not use if package is damaged and/or protection caps are detached. Keep dry. Store at controlled room temperature. Keep away from sunlight.
- Use aseptic technique.
- Single use only.

MORE INFORMATION:

- Length is approximate
- Approximate contained volume 0.35 mL
- Gravity flow rate 2,010 mL/hr
- Sterile and nonpyrogenic fluid path
- Fluid Path Material: PVC (Not made with DEHP) / PP (breather caps) / Copolyester (luer-locks) / MABS, Silicone (Needless Connector)

CONTRAINDICATIONS:

- Materials of the the extension set does not provide light protection.

PRIMING:

Priming must be completed before connecting to patient.

- Connect a fully primed infusion set or syringe to extension set female luer
- Prime K-Zero Needleless Connector
- Tighten all connections
- Clamp set
- Swab patient's access site or other luer connection before connecting to patient per facility protocol
- Remove end cap and attach to patient luer

SET REMOVAL:

- Stop infusion. Close slide clamp, disconnect the set from IV access and container in accordance with facility protocol. Replace per facility protocol or CDC guidelines. Discard biohazard waste according to facility protocol.

Manufactured For:

Fresenius Kabi AG
Else-Kröner-Str.1
61352 Bad Homburg, Germany
www.fresenius-kabi.com

Distributed in USA by:

Fresenius Kabi USA, LLC
Three Corporate Drive
Lake Zurich, Illinois 60047
www.fresenius-kabi.us
Tel.: 1-800-933-6925

N. Code No. : HR/Drugs 737-B(H)

10.5520.00

10.5966.01