

## White Male or Female Dead-End Cap

### FOR PARENTERAL FLUIDS

---

#### CAUTIONS:

- Do not use if package is damaged and/or protection caps are detached. Keep dry. Store at controlled room temperature. Keep away from sunlight.
- Use aseptic technique.
- Single use only.

#### MORE INFORMATION:

- Combi-Lock Cap, White PP

#### CONTRAINDICATIONS:

- Materials of the the gravity administration set does not provide light protection.

#### PRIMING:

##### **Priming must be completed before connecting to patient.**

- Connect to stopcock port
- Tighten white cap
- Before use, ensure that the stopcocks are properly primed to remove any air bubbles
- Remove white cap
- Replace white cap per hospital policy
- Swab patient's access site or other luer connection before connecting to patient per facility protocol

#### SET REMOVAL:

- Stop infusion. Close slide clamp, disconnect the set from IV access and container in accordance with facility protocol. Replace per facility protocol or CDC guidelines. Discard biohazard waste according to facility protocol.

**Manufactured For:**  
Fresenius Kabi AG  
Else-Kröner-Str.1  
61352 Bad Homburg, Germany  
[www.fresenius-kabi.com](http://www.fresenius-kabi.com)

**Distributed in USA by:**  
Fresenius Kabi USA, LLC  
Three Corporate Drive  
Lake Zurich, Illinois 60047  
[www.fresenius-kabi.us](http://www.fresenius-kabi.us)  
Tel.: 1-800-933-6925

N. Code No. : HR/Drugs 737-B(H)

10.5520.00

10.5964.01